
Cat, Dust mite & Grass pollen Self-test

Ref. A5035
Self-test that detects abnormally high levels of specific IgE in the blood in response to an allergic reaction to cat, dust mites, and/or grass pollen.

Carefully read the instructions before performing the self-test, to familiarize yourself with the different components and understand how the test should be conducted. If the result is positive, we recommend that you seek further advice from healthcare professionals. If you suspect allergies or experience symptoms, always seek medical advice regardless of the results of this self-test.

1. Test indicator
2. Activator
3. Interpretation window
4. Sample well
5. Dilution well
6. Pipette
7. Lancet
8. Diluent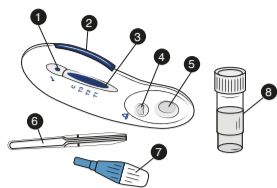

Content:
To facilitate the sampling process, we recommend having access to tissue paper and a clock.

A aluminium bag containing:
- 1 Multitest for Cat, Dust Mite & Grass Pollen
- 1 drying agent (not for use)

A transparent bag containing:
- 1 lancet
- 1 pipette
- 1 diluent
- 1 manual

1. Basics about this self-test
Quicktest Cat, Mite & Grass Pollen self-test is a simple, safe, and reliable way to detect high levels of antibodies (IgE) in blood specifically related to Egg Allergy. High levels of this specific IgE are associated with Egg Allergy. This self-test requires only a small amount of capillary blood easily taken with a provided lancet with a quick and simple finger prick.


Quicktest Cat, Mite & Grass Pollen self-test has shown good comparative results against laboratory tests, however, this self-test should not be used for diagnosis but should be used for a preliminary indication, diagnosis should always be made by a licensed physician. This self-test has no age limit, but a self-test is not suitable for children under the age of 3, not based on reliability, but on the procedure of blood collection. If your child is under 3 years of age, you should contact healthcare professionals.

The self-test can be used even if you are taking medication, and you do not need to have an ongoing allergic reaction for the test to be reliable. Do not ingest. Keep this IVD product out of reach of children and animals. The outer packaging should be sorted as cardboard/corrugated board, while the rest should be sorted as residual waste.

2. Preparation for the test
The self-test should be at room temperature during testing. If it has been exposed to cold, it is important to let the test sit at room temperature for about 2 hours first. Wash your hands with soap in warm water, rinse thoroughly. Dry your hands with a **CLEAN towel or paper**. It is important that your hands are warm, as it facilitates blood collection. Then open the outer packaging and place all components on a **CLEAN table surface**. Open the bags as follows:
• Transparent bag containing lancet, pipette, and diluent.
• Aluminium bag containing test cassette.


• **NOTE! DO NOT pull out the blue part (ACTIVATOR) of the test cassette, this step comes later.**

3. Sampling
With a simple prick using the included lancet, approximately one drop of blood should be collected into a pipette. If the blue tip of the lancet is broken, the lancet should **NOT** be used. Read more about this step below.
• Start by massaging the finger that will be pricked, this increases blood circulation and facilitates blood collection.
• Twist and pull off the blue top of the lancet.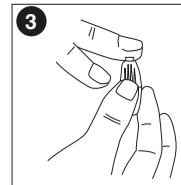

- Press the white narrow tip of the lancet **FIRMLY** against the finger.
- Remove the first drop of blood with a clean tissue.

4. Blood collection
1. Massage the finger to get a large drop of blood, press the finger with the hand that was not pricked - this facilitates blood collection.
2. The pipette is now gently placed against the blood droplet, and capillary action will draw the blood up. Only allow the amount of blood needed to flow up to the marking (line) on the pipette. If more blood is needed, massage the finger again. Try to avoid getting air bubbles into the pipette. Note that this step may take up to 2 minutes to complete. If you are not able to get the blood to flow into the pipette, do the following:
Set the pipette aside, continue to massage the finger until a drop has formed - if possible, let the drop fall freely into the test well (hole on the test cassette marked with a blood droplet). If the drop does not want to leave the finger, let the drop touch the test well and flow down. Wait for about 1 minute to let the blood be absorbed into the test well.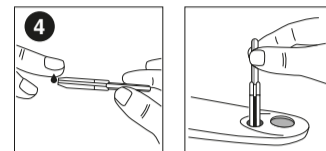


MANUAL SELF-TEST CAT, DUST MITE & GRASS POLLEN

EN

5. Addition of diluent
Remove the cap from the diluent, pour the entire contents into **the BIG hole (which is next to the test hole) and WAIT for 15 minutes. DO NOT TOUCH THE TEST CASSETTE!**

6. Read results
After 15 minutes: Pick up the test cassette and pull out the blue part on the side (**ACTIVATOR**) located next to the test window, pull it out until the entire test window is visible and a click is heard.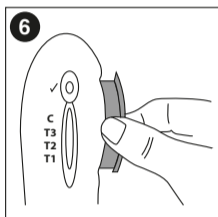


REPOSITION the test cassette on a flat surface. The self-test is now activated. If high levels of IgE are present in your blood, a positive result can be read in as little as 15 minutes. However, we recommend waiting for 30 minutes to read the result from when the activator was pulled out.

7. Interpret results
A pink or red line **MUST** appear at C (control). This indicates that the self-test is functioning correctly.

T1 = Cat. T2 = Dust mite. T3 = Grass pollen.

Positive result for Cat:
A pink or red line appears at the **T1** symbol. The strength or intensity of the line has no impact, if a line is discernible, it should be interpreted as a line. This means that elevated levels of IgE antibodies have been detected in your blood for this specific allergen.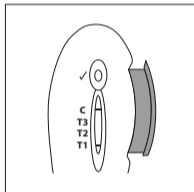




Positive result Dust mite:
A pink or red line appears at the symbol **T2**. The strength or intensity of the line is irrelevant, if a line can be distinguished it should be interpreted as a line. This means that elevated levels of IgE antibodies have been detected in your blood for this specific allergen.

Positive result Grass pollen:
A pink or red line appears at the symbol **T3**. The strength or intensity of the line has no impact, if a line can be distinguished, it should be interpreted as a line. This means that elevated levels of IgE antibodies have been detected in your blood for this specific allergen.

Note that this self-test can result in a positive result for more than one allergy.

Negative result
No pink or red line appears at T1, T2 or T3. This means that no elevated levels of IgE allergen antibodies have been detected in your blood for this specific allergen.

Failed result
If no line appears at C (Control), the test has failed and should be discarded.

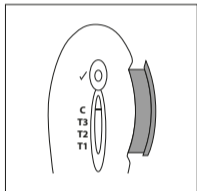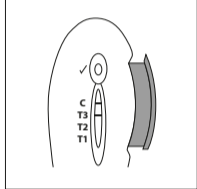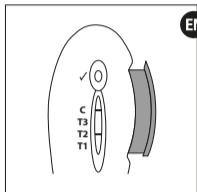IMPORTANT!

Do not make any medical decisions without first consulting healthcare.
To avoid the spread of infection, it is recommended to gather all components in the original package, seal it, and dispose of it as residual waste.

False positive and false negative
High levels of IgE are an established and reliable indication of allergy. However, it is important that you have previously experienced symptoms or currently have symptoms for the test to be reliable. Normal levels of IgE cannot exclude the possibility that you may develop allergies in the future. Some individuals may have elevated levels of specific IgE without experiencing symptoms of egg allergy, while in some cases mild discomfort may be experienced, and in others, more severe discomfort may be experienced. Additionally, proactive antibodies such as IgG may have an impact on various symptoms, but this varies by individual.

Note that if you suffer from rheumatism, this test may show a positive result even if you do not have any allergy.

	Test unit: Imutest Limited, Wilmslow, Cheshire, SK9 5DQ, UK	Directive 98/79/EC		0344
	Advena Ltd, Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta			
	Lancet: Becton & Dickinson Company, 1 Becton Drive, NJ, 07417, USA	Directive 93/42/EC		0050
	Becton Dickinson UK Ltd, Oxford UK			
	Pipette: Safe-Tec Clinical Products 142 Railroad Drive, Ivyland, PA 18974, USA	Directive 98/79/EC		
	CEpartner4U BV, 39151 DB Maarn, NL			

	Read instructions before use		For in vitro diagnostic use		Do not reuse
	Store between +4° et +25°C		Batch number		Expiry date
	Manufacturer	MD- 080020 m5 c - Revised 2022/04			

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