
Milk Protein Allergy Self-test

Ref. A5008
Self-test that detects abnormally high levels of specific IgE in blood during an allergic reaction.

Carefully read the instructions before performing the self-test, to familiarize yourself with the different components and understand how the test should be conducted.

- 1. Test indicator
2. Activator
3. Interpretation window
4. Sample well
5. Dilution well
6. Pipette
7. Lancet
8. Diluent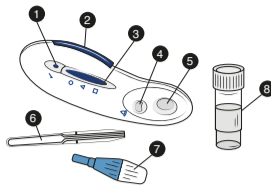

Content: To facilitate the sampling process, we recommend having access to tissue paper and a clock.

- A aluminium bag containing:
- 1 milk allergy test
- 1 drying agent (not for use)
A transparent bag containing:
- 1 lancet
- 1 pipette
- 1 diluent
- 1 manual

EN

MANUAL SELF-TEST MILK PROTEIN ALLERGY


EN

5. Adding dilution solution

Remove the cap from the diluent, pour the entire contents into the BIG hole (which is next to the test hole) and WAIT for 15 minutes. DO NOT TOUCH THE TEST CASSETTE!

6. Read results

After 15 minutes: Pick up the test cassette and pull out the blue part on the side (ACTIVATOR) located next to the test window, pull it out until the entire test window is visible and a click is heard.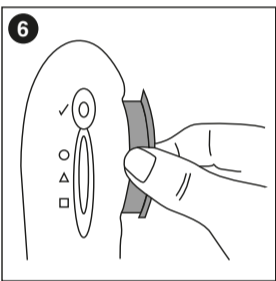

7. Interpret results

A pink or red line MUST appear besides the symbol Δ This indicates that the self-test is working correctly. No line should appear besides the symbol ○.

Positive result


A pink or red line appears besides the symbol □. The strength or intensity of the line does not matter, if a line can be distinguished it should be interpreted as a line.

Negative result

Only a pink or red line has appeared besides the symbol Δ. This means that no elevated levels of IgE allergen antibodies have been detected in your blood for this specific allergen.

Failed result

If no line appears besides the Δ, the test has failed and should be discarded.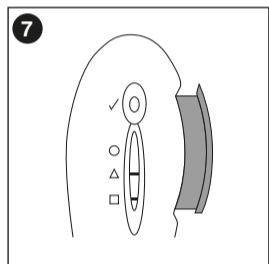

EN

1. Basics about this self-test

Quicktest Milk Protein Allergy self-test is a simple, safe, and reliable way to detect high levels of antibodies (IgE) in blood specifically related to Milk Protein Allergy.

Quicktest Milk Protein Allergy self-test has shown good comparative results against laboratory tests, however, this self-test should not be used for diagnosis but should be used for a preliminary indication.

The self-test can be used even if you are taking medication, and you do not need to have an ongoing allergic reaction for the test to be reliable.

2. Preparation for the test

The self-test should be at room temperature during testing. If it has been exposed to cold, it is important to let the test sit at room temperature for about 2 hours first.

NOTE! DO NOT pull out the blue part (ACTIVATOR) of the test cassette, this step comes later.

IMPORTANT!

Do not make any medical decisions without first consulting healthcare.

To avoid the spread of infection, it is recommended to gather all components in the original package, seal it, and dispose of it as residual waste.

False positive and false negative

High levels of IgE are an established and reliable indication of allergy. However, it is important that you have previously experienced symptoms or currently have symptoms for the test to be reliable.


Note that if you suffer from rheumatism, this test may show a positive result even if you do not have any allergy.

Table with columns for Test unit, Lancet, and Pipette, including manufacturer names like Imutest Limited and Becton Dickinson Company.

EN


3. Sampling

With a simple prick using the included lancet, approximately one drop of blood should be collected into a pipette. If the blue tip of the lancet is broken, the lancet should NOT be used.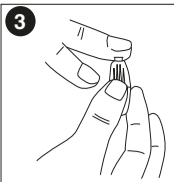

4. Blood collection

1. Massage the finger to get a large drop of blood, press the finger with the hand that was not pricked - this facilitates blood collection. 2. The pipette is now gently placed against the blood droplet...

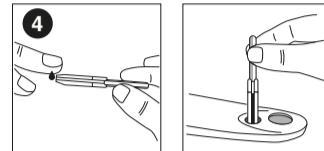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Advena Ltd, Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta
Becton Dickinson UK Ltd, Oxford UK